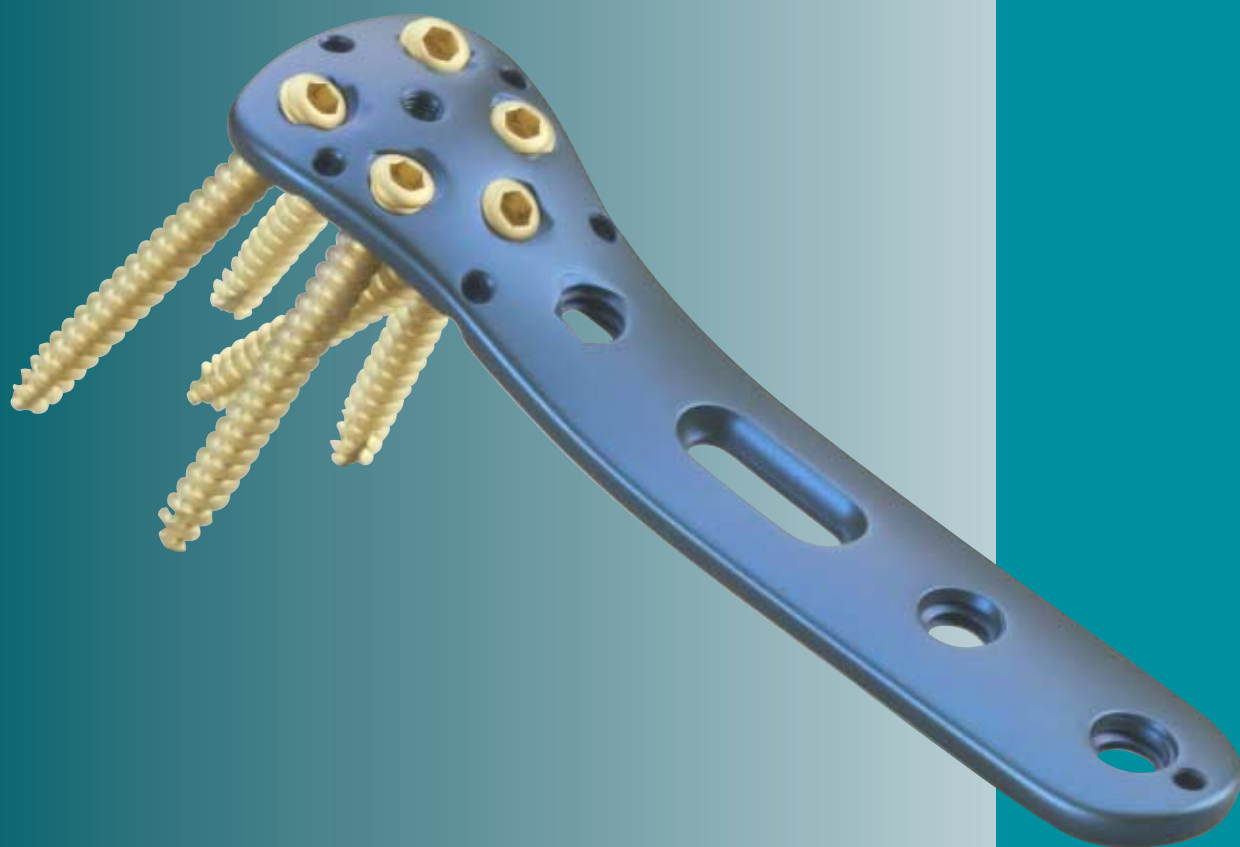
WINKELSTABILE

PROXIMALE HUMERUSPLATTE

Fixed-angle proximal humerus plate

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Fracture of the head of the humerus is a problem fracture. Primary joint replacement has not produced the desired solution and is now only recommended in individual cases or in the elderly patient.

One problem in the head-preserving operation is secure fixation of the often osteoporotic bone fragments. Because of their diverging position extending into the subchondral region, the 5 angle stable screws for the head that are anchored in the plate enable fixed spatial stability to be obtained and thus allow early functional aftercare. The course of the screws overall is in an oblique cranial direction so that the plate is located somewhat more distally. In conjunction with the slight thickness of the plate, this prevents plate impingement. Additional holes are incorporated in the head of the plate to which tension banding of the greater or lesser tuberosity, if required, can be fixed easily and securely. The possibility of angle stable fixation of the shaft screws also allows periosteum-sparing operating.

Präoperativ

Preoperative

Indikation

Indication

- Rekonstruierbare Humeruskopffrakturen
- Instabile subkapitale Frakturen

- Reconstructable fractures of the head of the humerus
- Unstable subcapital fractures

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Postoperative brief immobilisation, early commencement of passive exercises. Gradual transition to active exercising after about 3 weeks.

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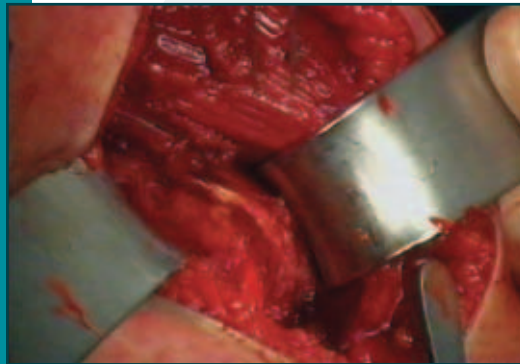
"Beach chair" position or supine with the upper body slightly elevated at the edge of the operating table. Image intensifier control must be available.

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Deltoideo – pectoral access.
(gentle dissection down to the proximal humerus head/shaft region causing as little trauma as possible)

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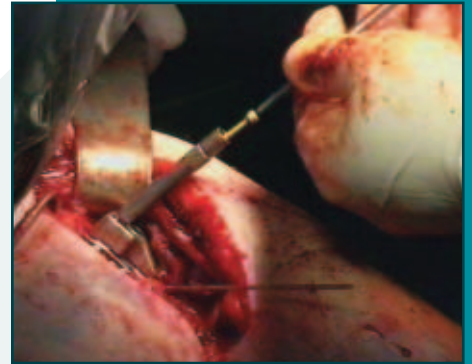
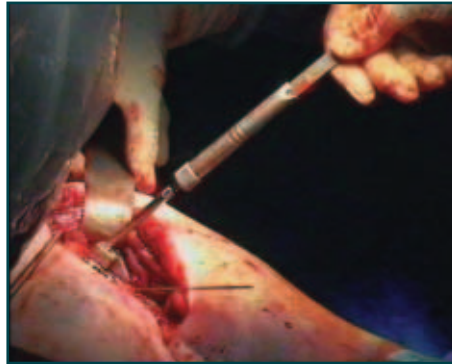
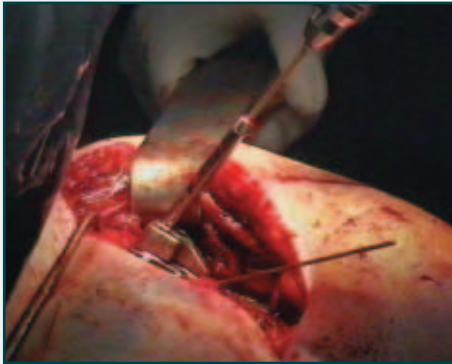
Exposure and reduction of the fracture (initially in the head/shaft region, then the tuberosity fragments). Temporary fixation with K-wires if necessary.

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Apply the plate with the guide block screwed on. Drill and insert a conventional screw in the long hole. Fine reduction with regard to height and rotation under image intensifier control.

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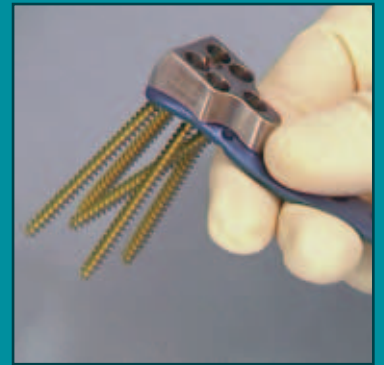
Drill for the angle stable screws through the screwed-on drill guide. Determine the length with the screw gauge with the drill guide in place (without protective sleeve on the screw gauge).

Exchange the drill guide for the screw bush, which enables the 4.0 mm diameter angle stable cancellous screws to be inserted easily and securely.

In the same way, fill the other plate holes in the head region; the screws should extend to close to the boundary of the head without perforating it (exact image intensifier control in 2 planes).

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The plate can now be fixed to the shaft optionally with angle stable or conventional 3.5 mm diameter cortical screws (use the drill bush for assistance if using angle stable screws).

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
In the case of complex fractures of the head of the humerus with avulsion of the greater or lesser tuberosity, additional fixation with tension band sutures through the lateral plate holes is recommended.

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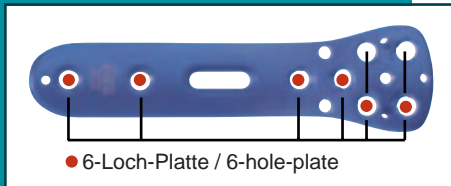Image intensifier control in 2 planes, vacuum drain, wound closure in layers.

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Cancellous screws, D 4,0 mm (conical head thread, self-tapping)

Bestell-Nr. Stahl Code N° steel	Bestell-Nr. Titan Code N° titanium	Länge Length
2.147.34	3.147.34	34 mm
2.147.36	3.147.36	36 mm
2.147.38	3.147.38	38 mm
2.147.40	3.147.40	40 mm
2.147.42	3.147.42	42 mm
2.147.44	3.147.44	44 mm
2.147.46	3.147.46	46 mm
2.147.48	3.147.48	48 mm
2.147.50	3.147.50	50 mm
2.147.52	3.147.52	52 mm
2.147.54	3.147.54	54 mm
2.147.56	3.147.56	56 mm

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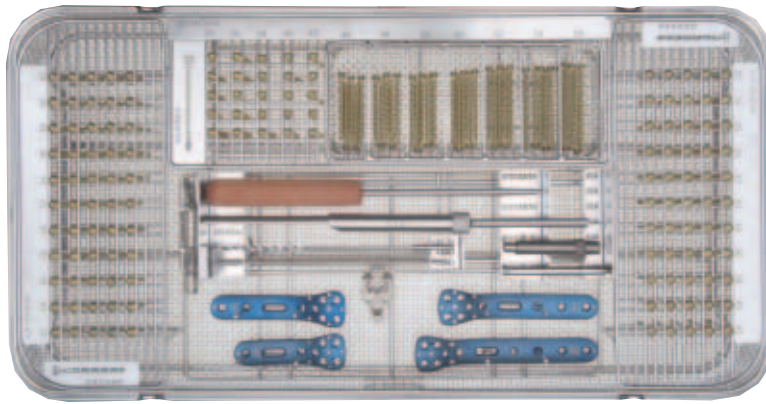
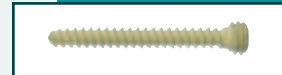
No part of this publication may be reproduced, stored in a retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying, recording or otherwise without the prior permission of the publisher.

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OSTEOSYNTHESE GmbH
KÖNIGSEE IMPLANTATE UND INSTRUMENTE ZUR OSTEOSYNTHESE GmbH